

AKASPRAY – MEDICAL DEVICE SURFACE DISINFECTANT

DESCRIPTION OF PRODUCT:

Alcohol based spray surface disinfectant

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Limpid	DETC03
Colour	-	Clear	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	6-8	DETC09
Density (20°C)	g/cm ³ , 20°C	0,95 ± 0,05 g/cm ³	DETC11
Solubility in water	-	Soluble	DETC13
Isopropyl alcohol	W/W	10%±15	DETC15
Ethanol	W/W	50%±15	DETC17

*Confirmed by Detrox Quality Control Department.

Microbiological Characteristics

Parameter	M.U.	Specifications	Methods of Analysis
Escherichia coli NCTC 10538	cfu/ml	30 sn	EN 13727, EN1276
Pseudomonas aeruginosa ATCC 15442	cfu/ml	30 sn	EN 13727, EN1276
Staphylococcus aureus ATCC6538	cfu/ml	30 sn	EN 13727, EN1276
Enterococcus hirae ATCC 10541	cfu/ml	30 sn	EN 13727, EN1276
Mycobacterium avium ATCC 15769	cfu/ml	5 min	EN 14348
Mycobacterium terrae ATCC 15755(Strain, W45)	cfu/ml	5 min	EN 14348
Candida albicans ATCC 10231	cfu/ml	30 sn	EN 13624
Aspergillus Niger ATCC 16404		30 sn	EN 13624

AKASPRAY – MEDICAL DEVICE SURFACE DISINFECTANT

Adenovirus	cfu/ml	1 min	EN 14476
Poliovirus	cfu/ml	1 min	EN 14476
Norovirus	cfu/ml	1 min	EN 14476

INTENDED USE OF PRODUCT:

Usage Area

- Akaspray is suit for using at surgery rooms, intensive cares, dental units, medical devices surfaces etc...

Ingredients

- %50 Ethanol, %10 Isopropyl alcohol, Quaternary ammonium compounds

Usage form

- It is sprayed from 30 cm distance to area. Wait 1 min. to effect. It is possible to be sprayed to a cloth and then apply by wiping. Product makes cleaning and disinfection. Switch off electronic devices before applying their surfaces.

Microbiological effect

- Bactericidal, fungicidal virucidal, tuberculocidal.

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.
Mavi Sokak No:10 Kat:1 Beşiktaş, İstanbul
No: T.Emniyeti - İstanbul - 34398/1191
Tel: 0212 659 77 62 - 0212 659 77 63