

## DETROSEPT AF – SPRAY SURFACE DISINFECTANT

### ***DESCRIPTION OF PRODUCT:***

Alcohol based spray surface disinfectant

### ***SPECIFICATION OF PRODUCTS:***

#### **Chemical-Physical Characteristic**

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Limpid	DETC03
Colour	-	Clear	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	6-8	DETC09
Density (20°C)	g/cm <sup>3</sup> , 20°C	0,95 ± 0,05 g/cm <sup>3</sup>	DETC11
Solubility in water	-	Soluble	DETC13
Isopropyl alcohol	W/W	20%±15	DETC15
Ethanol	W/W	10%±15	DETC17

\*Confirmed by Detrox Quality Control Department.

#### **Microbiological Characteristics**

Parameter	M.U.	Specifications	Methods of Analysis
Staphylococcus aureus ATCC 6538	cfu/ml	Effective in 1 min.	EN 13727, EN 14561
Pseudomonas aeruginosa ATCC 15442	cfu/ml	Effective in 1 min.	EN 13727, EN 14561
Enterococcus hirae ATCC 10541	cfu/ml	Effective in 1 min.	EN 13727, EN 14561
Mycobacterium avium ATCC 15769	cfu/ml	Effective in 5 min.	EN 14348
Mycobacterium terrae ATCC 15755(Strain, W45)	cfu/ml	Effective in 5 min.	EN 14348
Candida albicans ATCC 10231	cfu/ml	Effective in 1 min.	EN 13624, EN 14562
BVDV strain NADL ATCC-VR-534	cfu/ml	Effective in 5 min.	EN 14476
Vacinia virus strain Elstree CAMP V-160	cfu/ml	Effective in 5 min.	EN 14476

## **DETROSEPT AF – SPRAY SURFACE DISINFECTANT**

### ***INTENDED USE OF PRODUCT:***

#### **Usage Area**

- Detrosept AF is suit for using at surgery rooms, intensive cares, dental units, medical devices surfaces etc...

#### **Ingredients**

- %10 Ethanol, %20 Isopropyl alcohol, Quaternary ammonium compounds

#### **Usage form**

- Detrosept AF is sprayed from 30 cm distance to area. Wait 1 min. to effect. It is possible to be sprayed to a cloth and then apply by wiping. Product makes cleaning and disinfection. Switch off electronic devices before applying their surfaces.

#### **Microbiological effect**

- Bactericidal, fungicidal in 1 min, virucidal and tuberculocidal in 5 min.

#### **First aid Informations**

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL  
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.  
Atatürk Mahallesi Adıgözü Meydanı Caddesi  
No: 7 Etiler - Beşiktaş / İstanbul - Türkiye  
Tel: 0 212 659 77 62  
Fax: 0 212 659 77 63 - Detrox (T.C.) V.D. : 283 060 6762