

## DETROSAN AF – SPRAY SURFACE DISINFECTANT

### ***DESCRIPTION OF PRODUCT:***

Alcohol free, QAC and Amine based spray surface disinfectant

### ***SPECIFICATION OF PRODUCTS:***

#### **Chemical-Physical Characteristic**

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Limpid	DETC03
Colour	-	Colourless	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	6-9	DETC09
Density (20°C)	g/cm <sup>3</sup> , 20°C	1,04 ± 0,05 g/cm <sup>3</sup>	DETC11
Solubility in water	-	Soluble	DETC13
Benzalconium Chloride	W/W	0,45%±%15	DETC15
DDAC	W/W	0,3%±%15	DETC17
Dodecylamine	W/W	0,12%±%15	DETC19

\*Confirmed by Detrox Quality Control Department.

#### **Microbiological Characteristics**

Parameter	M.U.	Specifications	Methods of Analysis
Escherichia coli NCTC 10538	cfu/ml	Effective in 1 min.	EN 13727
Staphylococcus aureus ATCC 6538	cfu/ml	Effective in 1 min.	EN 13727
Pseudomonas aeruginosa ATCC 15442	cfu/ml	Effective in 1 min.	EN 13727
Enterococcus hirae ATCC 10541	cfu/ml	Effective in 1 min.	EN 13727
Mycobacterium avium ATCC 15769	cfu/ml	Effective in 1 min.	EN 14348
Mycobacterium terrae ATCC 15755(Strain, W45)	cfu/ml	Effective in 1 min.	EN 14348
Candida albicans ATCC 10231	cfu/ml	Effective in 1 min.	EN 13624

**DETRO HEALTHCARE KİMYA SANAYİ A.Ş.**

☎ 212 659 77 62 📠 212 659 77 63

✉ info@detrox.com.tr - www.detrox.com.tr

## DETROSAN AF – SPRAY SURFACE DISINFECTANT

Aspergillus niger ATCC16404	cfu/ml	Effective in 1 min.	EN 13624
Adeno virüs ATCC VR-5	cfu/ml	Effective in 1 min.	EN 14476
Poliovirüs ATCC VR-192	cfu/ml	Effective in 1 min.	EN 14476
Murine Norovirüs PTA-5935	cfu/ml	Effective in 1 min.	EN 14476

### **INTENDED USE OF PRODUCT:**

#### **Usage Area**

- Detrosan AF is suit for using at surgery rooms, intensive cares, dental units, medical devices surfaces etc. It can be used on incubators.

#### **Ingredients**

- Bis (3-aminopropyl) dodecylamine, Didecyldimethylammonium chloride, Benzalkonium chloride

#### **Usage form**

- Detrosan AF is sprayed from 30 cm distance to area. Wait 1 min. to effect. It is possible to be sprayed to a cloth and then apply by wiping. Product makes cleaning and disinfection. Switch off electronic devices before applying their surfaces.

#### **Microbiological effect**

- Bactericidal, fungicidal, virucidal, tuberculocidal in 1 min.

#### **First aid Informations**

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL  
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.  
Atatürk Mahallesi Adıyaman Caddesi  
No: 7 Etiler - Beşiktaş / İstanbul / Türkiye  
Tel: 0 212 659 77 62  
Fax: 0 212 659 77 63 - Detrosan V.D. : 283 060 8762