

# DETRO MULTICARE – INSTRUMENT CLEANING AND MAINTENANCE SOLUTION

## ***DESCRIPTION OF PRODUCT:***

Cleaning solution, rust remover, maintenance solution for medical instruments

## ***SPECIFICATION OF PRODUCTS:***

### Chemical-Physical Characteristic

Parameters	M.U	Specifications
Physical form	-	Liquid
Appearance	-	Limpid
Colour	-	Colourless
Odour	-	Characteristic
pH	pH unit, 20°C	1 - 3
Density (20°C)	g/cm <sup>3</sup> , 20°C	1,00 ± 0,1 g/cm <sup>3</sup>
Solubility in water	-	Soluble

\*Confirmed by Detrox Quality Control Department.

# DETRO MULTICARE – INSTRUMENT CLEANING AND MAINTENANCE SOLUTION

## ***INTENDED USE OF PRODUCT:***

### Product Properties

- Concentrate product, it should be diluted.
- It make completely maintenance of instruments, prevents corrosion.
- Effective cleaning property.
- It has corrosion inhibitors.
- 

### Usage Areas

- It is used for cleaning of rust layers, burnt and spots occurring after heating, oxidized parts and remnant causing rust on stainless steel instruments, laboratory and dental equipments.

### Usage form

- Solution is poured to a basin or ultrasonic device to dilute at %5 usage concentration (50 ml product in 1 lt water) and then immersed instruments.
- Wait 5-10 min. at 45-55 °C.
- At first, solution has limpid appearance.
- Solution should be renewed daily in case of high contamination situation.
- Rinse instruments after washing.

Effect time: 5-10 min. at ultrasonic usage

### Ingredients

Phosphoric acid, anionic tensides, nonionic tensides, corrosion inhibitors

### First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL  
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.  
Atatürk Mahallesi Akşeyh Mustafa, Çarşı:  
No: 7 Etiler - Beşiktaş / İstanbul / Türkiye  
Tic. Sic. No: 271167 / Şişli / İstanbul / Türkiye  
Tic. Sic. No: 271167 / Şişli / İstanbul / Türkiye