

DETRO FORTE – INSTRUMENT&ENDOSCOPE HIGH LEVEL DISINFECTANT

DESCRIPTION OF PRODUCT:

%2 glutaraldehyde based high level instrument and endoscope disinfectant

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Liquid	DETC01
Appearance	-	Limpid	DETC03
Colour	-	Clear	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20'C	3,5-4,5	DETC09
Density (20°C)	g/cm ³ , 20'C	1,02 ± 0,1 g/cm ³	DETC11
Solubility in water	-	Soluble	DETC13
Glutaraldehyde	W/W	2%±15	DETC15

*Confirmed by Detrox Quality Control Department.

Microbiological Characteristics

Parameter	M.U.	Specifications	Methods of Analysis
Staphylococcus aureus ATCC 6538	cfu/ml	Effective in 5 min.	EN 13727, EN 14561
Pseudomonas aeruginosa ATCC 15442	cfu/ml	Effective in 5 min.	EN 13727, EN 14561
Enterococcus hirae ATCC 10541	cfu/ml	Effective in 5 min.	EN 13727, EN 14561
Mycobacterium avium ATCC 15769	cfu/ml	Effective in 5 min.	EN 14563, EN 14348
Mycobacterium terrae ATCC 15755(Strain, W45)	cfu/ml	Effective in 5 min.	EN 14563, EN 14348
Candida albicans ATCC 10231	cfu/ml	Effective in 5 min.	EN 13624, EN 14562
Aspergillus niger ATCC16404	cfu/ml	Effective in 5 min.	EN 13624, EN 14562
Adeno virüs ATCC VR-5	cfu/ml	Effective in 5 min.	EN 14476+A1
Poliovirüs ATCC VR-192	cfu/ml	Effective in 5 min.	EN 14476+A1
Bacillus Subtilis ATCC 6633	cfu/ml	Effective in 15 min.	EN 13704

DETRO FORTE – INSTRUMENT&ENDOSCOPE HIGH LEVEL DISINFECTANT

INTENDED USE OF PRODUCT:

Usage Area

- Used for disinfection and cold sterilization of general surgical equipments, endoscopes and anesthesia equipments.

Ingredients

- % 2 Glutaraldehyde, auxiliary substances

Usage form

- Before usage of solution, precleaning of instruments should be made effectively.
- Pour 100 ml activator into 5 lt product drum. Close cover and shake 30 sn. then product is activ and ready.
- Pour solution into sink or machine that will be used.
- Immerse instruments into solutions thoroughly.
- After waiting effect time (15 min.), take out instruments and rinse with sterile water.
- Usage duration of solution is 28 days. In the case of intensive contamination on solution, it is recommended to renew.
- In the case of spattering or pouring to anywhere, there must wash with plenty of water immediately. Don't wait to dry.

Microbiological effect

Bactericidal, fungicidal, virucidal, tuberculoidal in 5 min, sporicidal in 15 min.

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.
Atatürk Mahallesi Akşirli Mh. Katlıca Cad. No: 7 Etiler
Beşiktaş / İstanbul / Türkiye
Tic. Sic. No: 274995 / Vergi Sic. No: 274995 / V.D. No: 28318640782