

The data presented herein is based on experiments and information believed reliable. However, we can make no guarantee on performance of, or results obtained through the use of the product herein described owing to varying conditions in laboratories and plants over which we have no control. Neither can any guarantee be given that the products or uses outlined will not infringe any existing patents.

## Health Care

# SANICIDE 5 PARFUME

## CONCENTRATED DETERGENT DISINFECTANT

### DESCRIPTION

- Sanicide 5 Parfume is a concentrated detergent/disinfectant for cleaning and contact disinfecting equipment in operating theatres and non-invasive medical devices.
- Sanicide 5 Parfume leaves a fresh, pleasant scent.

### WHERE TO USE

- Sanicide 5 Parfume is recommended for disinfecting floors, worktops, walls, furniture, bathrooms...etc.
- Sanicide 5 Parfume can be used for all surfaces in hospitals, clinics, retirement homes, dental and medical surgeries, industries and community units.

### HOW TO USE

- **For normal use:** Dilute Sanicide 5 Parfume to 0.25% (20ml dose in 8 lit water).  
For heavily soiled items, increase the concentration to 1%.
- Clean floors, walls, worktops, surfaces and bathrooms using a soaked cloth or by washing.
- If washing, leave the product to work for few minutes before removing the emulsion (e.g. water aspirator, Faubert brush).
- No need to rinse.

### INGREDIENTS

- Stabilized peroxides.
- Quaternary Ammonium (Benzalkonium Chloride).
- Pine essential oils.
- Non-ionic surfactants.

### TECHNICAL DATA

Colour	Amber
Odour	Pine
pH	8.1
Foam	No foam
Rinsing	Complete
Storage	Dry place
Fuming Capacity	Non-fuming

### SECURITY

Health	Moderate
Flammability	Insignificant
Reactivity	Insignificant
Shelf Life	4 years