

DETRO DENT CLEANING SOLUTION FOR DENTAL INSTRUMENTS

DESCRIPTION OF PRODUCT:

Cleaning solution for dental instruments, cement and alginate cleaner

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications
Physical form	-	Liquid
Appearance	-	Limpid
Colour	-	Colourless
Odour	-	Characteristic
pH	pH unit, 20°C	2 - 4
Density (20°C)	g/cm ³ , 20°C	1,00 ± 0,1 g/cm ³
Solubility in water	-	Soluble

*Confirmed by Detrox Quality Control Department.

DETRO DENT CLEANING SOLUTION FOR DENTAL INSTRUMENTS

INTENDED USE OF PRODUCT:

Product Properties

- Concentrate product, should be diluted before usage.
- Effective product for cement, alginat etc. cleaning in dental unites.
- Effective in 5-10 minutes at ultrasonic devices.

Usage Areas

- It is used for complete cleaning of glass, plastic and metal surgical instruments, dental equipments etc..

Effect time: 5-10 min

Usage form

- Solution is poured into a sink or ultrasonic cleaner machine and dilute with water to prepare at %3 concentration (30 ml product in 1 lt water). And then all instruments are immersed.
- Wait for effect time 5-10 min. at 45-55 °C
- After waiting effect time, take out instruments and rinse properly.

Ingredients

Cleaning agents, auxiliary substances, corrosion inhibitors

First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.
Muhür Mühürülmü AŞİTİ AÇIKLAMASI
No: 12/2019 - İstanbul - 15.06.2019
TUTANAK NO: 12/2019 - 15.06.2019