

DETRO ENZYM FOAM – INSTRUMENT CLEANING FOAM SPRAY

DESCRIPTION OF PRODUCT:

Foam cleaning spray for medical instruments

SPECIFICATION OF PRODUCTS:

Chemical-Physical Characteristic

Parameters	M.U	Specifications	Method of Analysis
Physical form	-	Foam	DETC01
Appearance	-	Foam	DETC03
Colour	-	White	DETC05
Odour	-	Characteristic	DETC07
pH	pH unit, 20°C	7 – 9	DETC09
Density (20°C)	g/cm ³ , 20°C	0,098 ± 0,1 g/cm ³	DETC11
Solubility in water	-	Soluble	DETC13

*Confirmed by Detrox Quality Control Department.

DETRO ENZYM FOAM – INSTRUMENT CLEANING FOAM SPRAY

INTENDED USE OF PRODUCT:

Product Properties

- Thanks to generated foam by ergonomic spray cap, solution covers instruments as homogeneous.
- Thanks to foam that is keeping its stability long time and that expanding on surface in form of preventing air contact, it provides to stay instruments moist long term (while gathering and carrying) and so, it prevents to dry of organic contaminations.
- Thanks to triple enzyme ingredient, it effects to dissolve blood, oil, protein, starch residues on surfaces.
- It provides saving of time while gathering and carrying instruments to central sterilization units due to that it is preventing to dry of contaminations and so, it helps to save time while pre-cleaning of instruments.
- Thanks to its effective solubility in water, it can be rinsed easily.
- It has corrosion inhibitors, so it doesn't cause corrosion on instruments.

Usage Areas

- Before disinfection, it is used for moistening with foam and cleaning (while gathering and carrying) of all kind of surgical instruments, medical equipments and endoscopes.

Usage form

- Firstly apply Detro Enzym Foam to floor of basin and cover all floor with Detro Enzym Foam.
- Gather instruments there.
- After gathering, cover top of all instruments with Detro Enzym Foam again and close cover of basin.
- Wait for necessary effect time and then take instruments out and rinse thoroughly.

Ingredients

- Amilase, lipase, protease, cellulase enzymes, surfactants, protective agents, corrosion inhibitory, stabilizers.

First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

Netice TONBİL
General Manager

DETRO HEALTHCARE KİMYA SAN. A.Ş.
Atatürk Mahallesi Adıgözü Marmara Caddesi
No: 7 Şişli - Beşiktaş / İstanbul / Türkiye
Tel: +90 212 659 77 62
Fax: +90 212 659 77 63