

# BACTINYL SPRAY MOUSSANT PARFUME READY TO USE PERFUMED DISINFECTANT DETERGENT FOAMING SPRAY



## BENEFITS

- Safe for surfaces even the most fragile
- Bactericidal, yeasticidal and virucidal
- Food contact agreement



Formule douce



## APPLICATION

Cleaning and disinfection by contact of non-invasive medical devices, instrumentation, medical equipment and equipment.

Medical device. CLASS IIA.

## INSTRUCTIONS FOR USE

Applied by spraying BACTINYL SPRAY MOUSSANT PARFUME. Respect the action time according to the efficiency desired. Let dry. Do not rinse except in case of contact with surfaces that may come into contact with foodstuffs.

Does not replace disinfection by soaking.

Complies with the decree of 19/12/13 amending the decree of 08/09/1999 concerning products for cleaning materials and articles intended to come into contact with foodstuffs.

## CHARACTERISTICS

Appearance :	liquid
Colour :	colourless
Perfume :	pine
Density :	= 1
pH :	low base
Expiration :	⌚ 2 years
Stockage :	4°C — 35°C



## MICROBIOLOGICAL ACTIVITY

**Bactericidal** : EN 13727 (dirtiness conditions – 5 min – 20°C), EN 14561 (dirtiness conditions – 5 min – 20°C)

**Yeasticidal** : EN 13624 (dirtiness conditions – 5 min – 20°C), EN 14562 (dirtiness conditions – 15 min – 20°C)

**Sporicidal** : EN 13704 (cleanliness conditions – 60 min – 20°C)

**Virucidal** : EN 14476 on PRV\*, BVDV\*, H1N1, Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 5 min – 20°C), Norovirus (dirtiness conditions – 15 min – 20°C)

\* Swine Pseudorabies Virus (PRV) and Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

## PACKAGING

750ml bottle + foaming spray

Réf. LB920870

## SECURITY

For professional use only.

For more information please refer to the MSDS.

