



## BENEFITS

- Disinfection of all rotating instruments (drills...)
- Bactericidal, yeasticidal and virucidal



Formule douce



## APPLICATION

Ready-to-use product for cleaning and disinfecting all types of rotating or ducted instruments (drills and endodontic instruments), medical devices.

CLASS IIB.

## INSTRUCTIONS FOR USE



BACTINYL® FRAISES is used pure cleaning and disinfecting all types of rotating dental instruments, drills, and medical devices.

BACTINYL® FRAISES is used by soaking.

Completely immerse the instrumentation as soon as possible after use.

Leave to act for 15 minutes. Rinse. To dry.

## CHARACTERISTICS PHYSICO-CHIMIQUES

Appearance :	liquid
Colour :	colourless
Perfume :	odourless
Density :	= 1
pH :	10 +/- 1
Expiration :	 2 ans
Storage :	 4°C - 35°C

## MICROBIOLOGICAL ACTIVITY

**Bactericidal** : EN 13727 (dirtiness conditions – 5 min – 20°C), EN 14561 (conditions de saleté – 5 min – 20°C),

**Yeasticidal** : EN 13624 (dirtiness conditions – 5 min – 20°C), EN 14562 (conditions de saleté – 15 min – 20°C),

**Virucidal** : EN 14476 on PRV\*, BVDV\*, H1N1, Herpès, Vaccinia virus, Rotavirus (dirtiness conditions – 5 min – 20°C), Norovirus (dirtiness conditions – 15 min – 20°C)

\* Swine Pseudorabies Virus (PRV) et Bovine Viral Diarrhea Virus (BVDV) are surrogate of HBV and HCV.

## PACKAGING

5L can

Réf. LB920470

## SECURITY

For professional use only.

For more information please refer to the MSDS.